Deutsch

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English

1. Important information - read prior to start-up

You have acquired a valuable RIESTER product manufactured in compliance with Directive 93/42/EEC for medical products and subject to continuous stringent quality control. Should you have any queries, please contact the Company or your RIESTER Agent who will be pleased to assist you. For addresses see last page of these Operating Instructions. The address of your authorised RIESTER Agent will be supplied to you on request. Please also note that correct and safe operation of products will only be guaranteed when RIESTER products and accessories are used throughout.

Significance of symbol on scales:
Caution, observe provided information

2. Purpose

Spirometers are devices for measurements of lung performance during a therapy. This product does not give values which are relevant for a diagnostic or a therapy.

3. Components (see figure)

- 3.1. Spirometer
- 3.2. Skala
- 3.3. Düse
- 3.4. Mundstücke
- 3.5. Ring

4. Start-up and operation

- 4.1. Connect end of the mouthpiece (3.4.) to nozzle (3.3.).
- 4.2. Hold spirometer in one hand and check that the needle is set to "0".
- 4.3. If the needle is not set to "0" correct by rotating the upper outer ring (3.5.) in clockwise or counterclockwise direction.
- 4.4. Breathe in deeply and sit up straight. When your lungs are full, insert mouthpiece (3.4.) between your lips. Then breathe out continuously (within 4 - 5 seconds). This is very important to achieve precise results
- 4.5. When you have exhaled completely, read precise value (3.2.).
- 4.6. Repeat this procedure 3 times. The maximum value of 3 measurements corresponds to your lung capacity.

Precautions!

- During the test, hold "Spirotest" in horizontal position. (If the unit is not held steady, the device cannot rotate uniformly and measured results will not be correct).
- When using the unit, the small holes at the side of the upper section must not be covered with your hand.
- The "Spirotest" is a precision instrument similar to a clock, and must be handled with care.
- Only your doctor is in a position to analyse the values exactly.

5. Instruction for care

General information

The goal of cleaning and disinfection of medical products is the protection of patients, users and third persons and conserving the value of the medical products.

On account of the product design and the used material, no defined limit of maximum processing cycles can be fixed. The lifetime of the medical products depends on their function and on an appropriate treatment of the devices. Before returning faulty products for repair they must have gone through the described reprocessing process.

Cleaning and disinfection

The spirometer can be cleaned outside with a humid cloth until optical cleanliness is given.
Wipe disinfection according to the regulations given by the disinfectant manufacturer. Only disinfectants with approved efficiency and in accordance with the national standards can be used.
After disinfection the device must be cleaned with a humid cloth to remove all possible deposits of disinfectants.

Caution!

If the unit is used regularly, water droplets and saliva will condense on the inside of the housing, impairing precision. After every measurement remove the lower section and clean and disinfected according to the upper description!
Never place the spirometer in liquid!
The items are not permitted for machine cleaning and sterilisation as they will damaged!

6. Spare parts

Item No. 11230 Package of 50 pcs. plastic mouthpieces
Item No. 11232 Package of 1000 pcs. carton mouthpieces

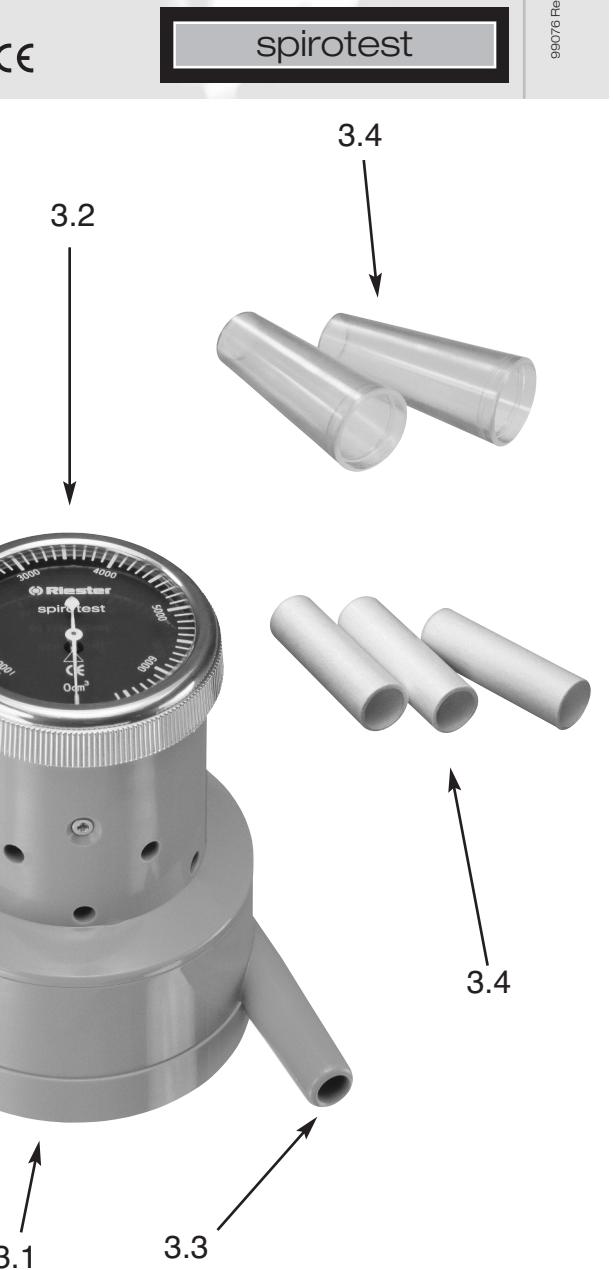
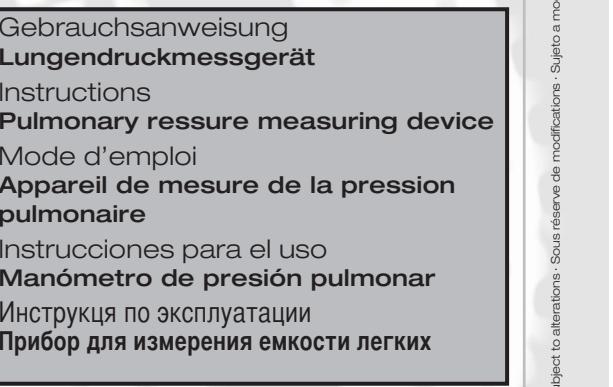
7. Maintenance

These instruments and their accessories do not require any specific maintenance.
Should an instrument have to be examined for any specific reason whatsoever, please return it to the Company or an authorised RIESTER dealer in your area. Addresses to be supplied on request.

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Warranty

This product has been manufactured under the strictest quality standards and has undergone a thorough final quality check before leaving our factory.

We are therefore pleased to be able to provide a warranty of

2 years from the date of purchase

on all defects, which can verifiably be shown to be due to material or manufacturing faults. A warranty claim does not apply in the case of improper handling.

All defective parts of the product will be replaced or repaired free of charge within the warranty period.

This does not apply to wearing parts.
For R1 shock-proof, we grant an additional warranty of 5 years for the calibration, which is required by CE-certification.

A warranty claim can only be granted if this Warranty Card has been completed and stamped by the dealer and is enclosed with the product.

Please remember that all warranty claims have to be made during the warranty period.

We will, of course, be pleased to carry out checks or repairs after expiry of the warranty period at a charge. You are also welcome to request a provisional cost estimate from us free of charge.

In case of a warranty claim or repair, please return the RIESTER product with the completed Warranty Card to the following address:

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