Deutsch

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English

- Important information to be read prior to first-time use**

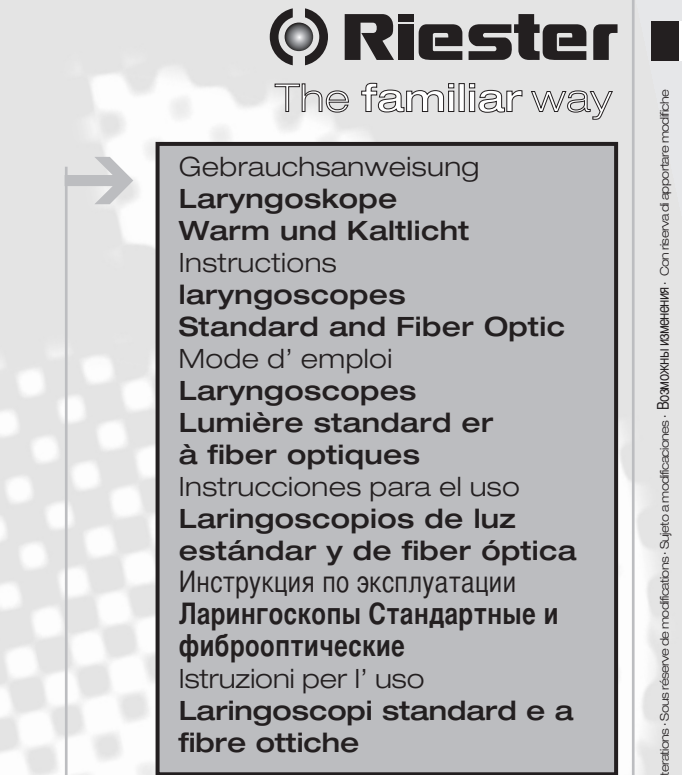
You have purchased a high-quality RIESTER product, which is manufactured in compliance with Directive 93/42/EEC for medical products and is subject to continuous stringent quality controls. This outstanding quality guarantees reliability and use. These instructions for use are applicable for fiber optic, standard illumination and ri-dispo laryngoscopes. Should you have any queries, please contact us or your authorised RIESTER dealer for assistance. Our address can be found on the last page of these instructions for use. The address of your RIESTER dealer will be supplied upon request. Please note that all instruments described in these instructions for use are only suitable for use by trained operators. Test the condition of the internal electrical energy source each time before use by turning on the lamp. Furthermore, please note that the correct and safe functioning of our instruments is only ensured when RIESTER instruments and accessories are used throughout.

2.1. Battery handles and first-time use

2.1. Purpose
The RIESTER battery handles described in these instructions for use serve to supply the lamps with power and to act as blade holders.

2.2. Battery handle range

a) Battery handle type C 28mm diameter, 2.5V for standard laryngoscopes, fiber optic laryngoscopes and ri-dispo
In order to operate these battery handles, you will require two commercially available 1.5 V type alkaline batteries (IEC standard designation LR14) or one 2.5 V or 3.5 V rechargeable battery (ri-accu®/ri-accu® L). The handle with the rechargeable RIESTER battery (ri-accu®/ri-accu® L)

can only be charged in a RIESTER ri-charger® /ri-charger® L charging base. Important: The handle with the Riester ri-accu® L can only be charged in the RIESTER ri-charger® L

b) Battery handle type AA 19mm diameter, 2.5V for standard laryngoscopes, fiber optic laryngoscopes and ri-dispo
In order to operate this battery handle, you will require two commercially available 1.5 V type AA Mignon alkaline batteries (IEC standard designation LR6) or a 2.5 V rechargeable battery (ri-accu®). Important: The handle with the Riester ri-accu® L can only be charged in the RIESTER ri-charger® L

c) Type C rechargeable battery handle 2.5 V or 3.5 V to charge from the mains 230 V or 120 V for fiber optic laryngoscopes and ri-dispo
The handle is available as a 2.5 V or 3.5 V model and can be ordered for 230 V or 120 V operation. Please note that the handle can only be used with the RIESTER ri-accu®/ri-accu® L

Safety:

	Application part type B
	Singel use <p>Repeated use could cause infection</p>

Remarks concerning electromagnetic compatibility:
At present, there is no indication that electromagnetic interactions with other equipment arise when these instruments are used as directed. Nevertheless, interference may not be completely ruled out under the increased influence of unfavourable field strengths, e.g. due to mobile telephones and radiological instruments.

2.3. Insertion and removal of batteries and rechargeable batteries

Handle types 2.2 a and b
Remove the handle cover from the bottom of the handle by twisting.
Insert either two batteries (as described in 2.2) or one 2.5 V or 3.5 V RIESTER rechargeable battery (ri-accu®/ri-accu® L) into the handle sleeve, with the positive terminals pointing towards the top of the handle.
The rechargeable battery (ri-accu®/ri-accu® L) is also marked with an arrow next to the plus sign, pointing in the direction in which it should be inserted into the handle. Twist the handle cover firmly back into place on the handle.
Batteries or rechargeable batteries (ri-accu®/ri-accu® L) are removed by first removing the battery handle cover. In order to remove the batteries or rechargeable battery, shake the handle slightly in a downward direction.
Prior to the first-time use of rechargeable handles, first place the rechargeable battery (ri-accu®/ri-accu® L) into the handle and charge up the handle in the RIESTER ri-charger® charging base.

The charging base includes additional instructions for use which have to be observed.

Important: The handle with the Riester ri-accu® L can only be charged in the RIESTER ri-charger® L

Handle types (2.2. c)

Prior to initial use of the plug-in handle, it should be charged for up to 24 hours in the mains socket. **Caution:** The plug-in handle (only for NiMH rechargeable batteries) must not be charged for longer than 24 hours.

Screw off the handle cover on the lower part of the handle. Depending on which handle you have purchased and for what voltage (see 2.2), insert the respective rechargeable batteries into the handle casing. Show you have any queries, please contact us or your authorised RIESTER dealer for assistance. Our address can be found on the last page of these instructions for use. The address of your RIESTER dealer will be supplied upon request. Please note that all instruments described in these instructions for use are only suitable for use by trained operators. Test the condition of the internal electrical energy source each time before use by turning on the lamp. Furthermore, please note that the correct and safe functioning of our instruments is only ensured when RIESTER instruments and accessories are used throughout.

Caution: The handle must never be in the mains socket when the rechargeable batteries are replaced! If you wish to replace the ri-accu®/ri-accu® L battery, unscrew the battery handle cover on the lower part of the handle counter clockwise.

Remove the ri-accu®/ri-accu® L battery from the battery handle by shaking down the handle downwards and lift. Insert the ri-accu®/ri-accu® L battery into the handle. Turn the battery handle clockwise, take care that the battery is inserted into the handle with the plus end towards the top of the handle; you will also find an arrow next to plus symbol which shows you the direction to insert into the handle. It is irrelevant in which direction 3.5 V rechargeable batteries are inserted. Screw the battery cover clockwise onto the handle.
Technical data: Either 230 V or 120 V

Warning:

- When placing the ri-accu®/ri-accu® L into the rechargeable handle, ensure that the battery handle cover is open.
- Should the instrument not be used for a longer period of time or whilst travelling, remove the batteries or the rechargeable battery from the handle.

- In the case of certain rechargeable batteries, the voltage may drop quickly when in use and this can lead to a rapid decrease in light intensity.
- Insert new batteries when the light intensity of the instrument is low, which can affect its efficiency in use.
- For maximum light yield we recommend that only new, high quality batteries are fitted when re-charging batteries.
- If you suspect that liquid or condensation has penetrated inside the handle, do not recharge it under any circumstances.
- Always ensure that the correct handle for standard and fiber optic blades is used. (Fiber optic handles are not interchangeable with a green ring).
- For ri-dispo please use the fiber optic handles

Disposal
Please note that batteries have to be disposed of separately. For details, ask your local authority and/or the responsible environmental consultant.

2.4. Attachment of laryngoscope blades

Attach the required laryngoscope blade to the runner on the upper part of the handle and pull up the front part of the blade until the back of the blade is in a line with the upper part of the handle. A click will be heard as it snaps into place. As the blade engages, the lamp will be turned on. The lamp is switched off as soon as the blade disengages from the snap attachment.

2.5. Care

Cleaning and/or disinfection
Battery handles may be cleaned from the outside with a moist cloth. Furthermore, the following disinfectants may be used for disinfection: Aldehydes (formaldehyde, glutaraldehyde, aldehyde derivatives), surfactants or alcoholic solutions.

When using such substances, observe the manufacturer's instructions closely. A soft, preferably lint-free cloth or Q-tips may be used as aids for cleaning or disinfecting.

For ri-dispo please use the fiber optic handles

Warning

Never immerse the handles in liquids. Ensure that no liquid penetrates inside the casing. Never clean batteries and rechargeable batteries with water or aqueous solutions.

Caution

The battery handle is not intended to be used for invasive procedures. Cleaning and/or disinfecting is fully adequate.

2.6. Spare parts

Rechargeable batteries
ri-accu® 2.5 V for handle AA, 2.5 V, 19 mm diameter
ri-accu® 2.5 V for F.O. and standard handles type C 2.5 V, 28 mm diameter
ri-accu® 3.5 V only for F.O. handles type C 3.5 V, 28 mm diameter
ri-accu® L rechargeable Li-Ion battery type AA 3.5 V
ri-accu® L rechargeable Li-Ion battery type AA 3.5 V
ri-accu® L rechargeable Li-Ion battery (plug in handle) 3.5 V

Quality batteries 1.5 V non-rechargeable
Article No. 10685 for handles type AA Packs of 4,
Article No. 10686 for handles type C 2.5 V Packs of 2.

Charging base ri-charger®

Article No. 10700 ri-charger® 2.5 V/3.5 V / 230 V, including wall bracket
Article No. 10701 ri-charger® 2.5 V/3.5 V / 120 V, including wall bracket
Article No. 10705 ri-charger® L 2.5 V/3.5 V / 230 V, including wall bracket
Article No. 10706 ri-charger® L 2.5 V/3.5 V / 120 V, including wall bracket

3. Laryngoscope blades

3.1. Purpose
The laryngoscopes described in these Instructions for Use have been manufactured for endotracheal intubation.

3.2. Replacement of lamp Standard blade (vacuum illumination):

Twist off the lamp at the front of the blade and replace it with a new lamp.
Fiber optic blades (Xenon and LED illumination):
a) Unscrew the lamp holder from the sleeve.
b) Unscrew the lamp holder at the knurl and remove.
c) Unscrew the lamp.
d) Screw the lamp holder back into the sleeve.
e) Screw back the lamp holder into the top part of the handle.
f) Screw the upper part of the handle into the handle sleeve.

3.3. Detaching/Inserting the light conductor ri-modul

This is possible by disengaging/engaging the light conductor from the side at the hinge connection of the blade.

3.4. Technical specifications of lamps

Lamp, small, Lamp, large,	2,7 V 300 mA 2,7 V 300 mA	average life standard 20 h average life standard 20 h
Lamp, XL 2.5 V fiber optics	680 mA	average life 20 h
Lamp, XL 3.5 V fiber optics	700 mA	average life 20 h
Lamp, LED 2.5 V fiber optics	120 mA	average life 20000 h
Lamp, LED 3.5 V fiber optics	280 mA	average life 20000 h

Warning:
Lamps in exposed places, which remain illuminated, can produce heat which is sufficient to burn human tissue.

3.5. Care

a) Predisinfection
Warning: The lamp should first be removed for standard blades! Always immerse instruments into a non-renewable disinfectant solution immediately after use. This is used to prevent the general migration of germs and to protect staff against contamination from germ-containing secretions which adhere to the blade and may constitute a hazard when it is subsequently cleaned. In addition, this prevents drying of any residual blood and secretions when the instrument is used as an antihorace.
Caution: The plug-in handle (only for NiMH rechargeable batteries) must not be charged for longer than 24 hours.

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